



## 3 Wheel Operating Instructions



**Evacusafe™**

innovation | manufacturing | training

UK Office 01256 33 27 23

Fax: 01256 33 28 23  
info@evacusafe.net  
www.evacusafe.net

Unit 9, Devonshire Business Park  
Knights Park Road,  
Basingstoke RG21 6XN

ECD02-June17-V1

**Evacusafe™**

innovation | manufacturing | training

## Home Emergency Transit Chair

**WARNING** – THE HETC IS DESIGNED FOR TRANSPORTING PEOPLE, IT MUST NOT BE LEFT UNATTENDED. ONCE IN USE IT MAY TIP BACK AND INJURE THE OCCUPANT.

THE CHAIR REQUIRES A MINIMUM OF TWO PEOPLE TO OPERATE.


### OPERATING INSTRUCTIONS



1. Remove chair from storage location and place on the ground in a suitable position.



2. Hold the handle extension and push in the red locking mechanism at rear of seat base, unfold until the chair locks into position.

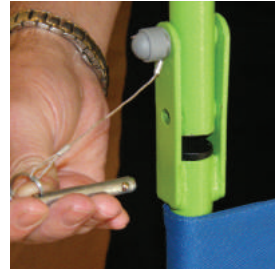

3. Unfold the handle extension and insert the safety pins on each side securely.



4. Using a suitable method transfer a patient into the chair. Fold down arms for security and comfort, place the persons feet on the footrest if required. Fasten the safety belt and move to the top of the stairs.



5. Using 2 people with 1 at the front and 1 at the back, move forward over the top step, allowing the 3 rotating wheels to balance, lower the chair and continue down each step. Once you are at ground level move to a place of safety.

### TO RETURN TO THE STORAGE POSITION –



1. Fasten seatbelt, fold away footrest and raise the arms.



2. Remove safety pins on each side and fold away handle extension.



3. Push in the red locking mechanism at rear of the seat base.



4. Fold down the chair into the storage position and return to storage location.

Remember to always be courteous, considerate and confident with your patient.